



Ventilator Circuit and NG Tubing Stabilizer

#701VCS/NG

PEDIATRIC

DIRECTIONS:

- 1 Slide end marked (B) around the patient's back and attach to end marked (C).
- 2 Lay ventilator circuit tubing on top of connected ends (B) and (C). Take end piece marked (D) and lay over ventilator circuit and attach to the opposite side of PART A.
- 3 Lay the NG/OG tubing on top of PART A. Take PART E and lay it over NG tubing connecting other end of PART E to PART A.

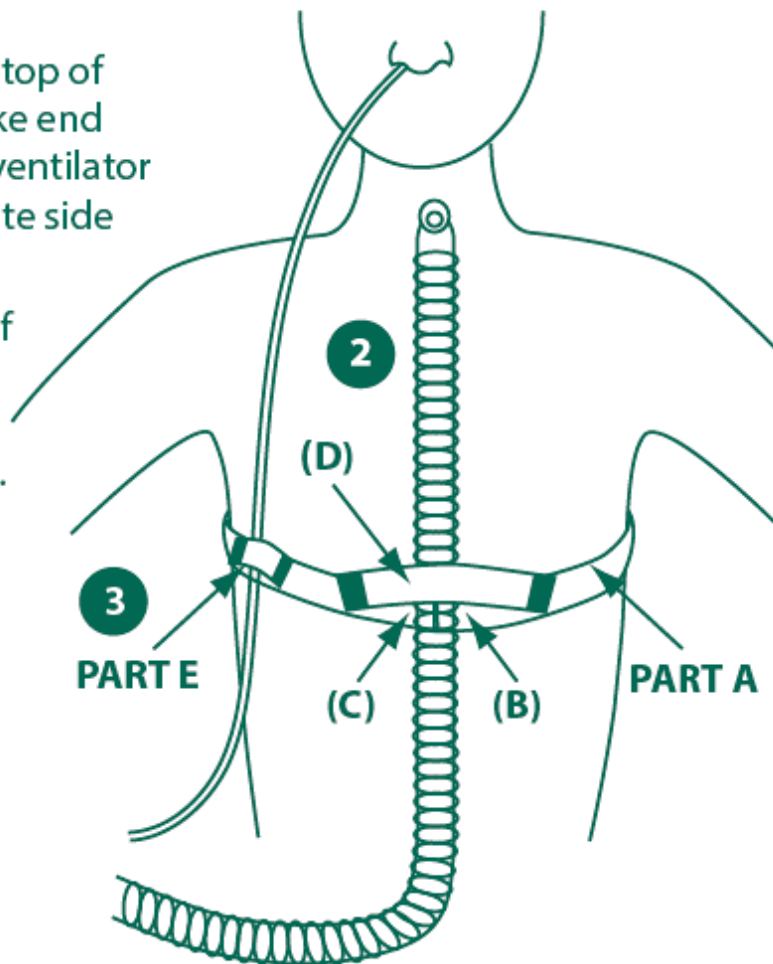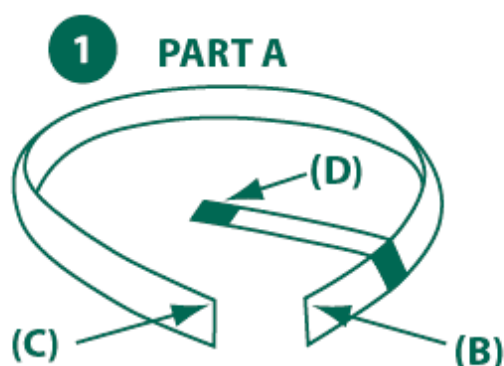

Patent Pending